

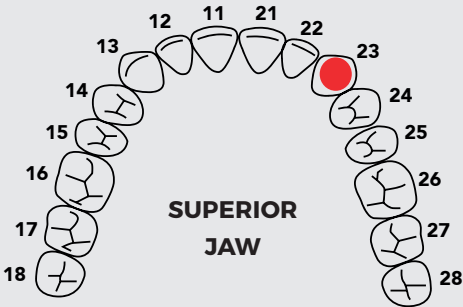
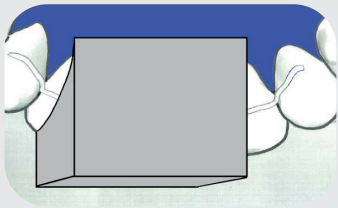


SMILERS® FIL INDIRECT GLUING PROTOCOL

COLOURED MARKINGS ON WIRES AT 23 AND 33

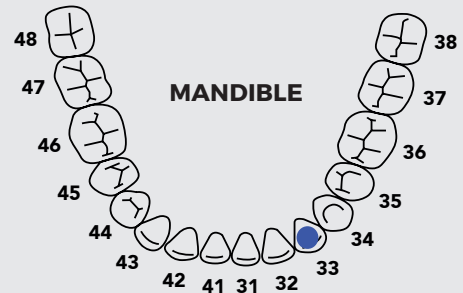

**ENDO-ORAL VIEW
WITH MAXILLARY WIRE**

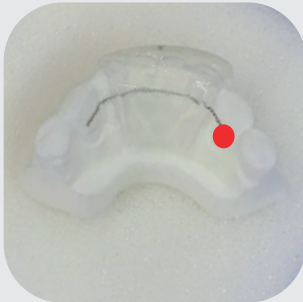

**ENDO-ORAL VIEW
WITH MANDIBULAR WIRE**



1
Clean

2
Etching
(GC Ortho Etching Gel advised)

3
Rinse / Dry

4
Wire positioning
with transfer key

5
Gluing the ends with glue having primer metal
(GC Ortho Connect Flow advised)

6
Remove transfer key

Polymerisation

7
Gluing teeth located
underneath the transfer key
with glue having primer metal
(GC Ortho Connect Flow conseillé)

Polymerisation