

smilers®

DIRECT BONDING PROTOCOL

Smilers® FIX with hooks



WARNING

SMILERS® FIX SHOULD ONLY BE PLACED
AFTER THE RETAINER PHASE

1. Try the retainer to check if it fits properly.
2. Perform the etching.
3. Rinse and dry. The enamel surfaces should have a chalky appearance.
4. Apply an adhesive.
5. Leave to work and dry.
6. Photopolymerisation.
7. Apply adhesive (the same as on the teeth) to the lower surface of the Smilers® FIX .
8. Dry and photopolymerisation.
9. Coat the lower surface of the retainer with composite or with bonding cement, and also apply to the teeth surfaces.
10. Gently place the retainer in the mouth and check that it fits properly.
11. Perform an express photopolymerisation on several spots, then remove the excess.
12. Perform a complete photopolymerisation on several spots, through the retainer.
13. Remove the hooks with a dental drill.
14. Clean any excess of composite or bonding cement with a drill.
If necessary, polish the retainer.



**Make sure you follow the protocol dedicated to the product you are using
(adhesive, composite or bonding cement).**